

EasySlide

SystemRoMedic

**Safe and
easy**

**Prevents
pressure
sores**

NEW!

*disposable EasySlide
see separate manual*

EasySlide was the very first SystemRoMedic transfer aid. Since 1984 it has been successfully used in hospitals and nursing homes around the world. Their own unique sliding material, ULF (Ultra Low Friction), on the inner surface of the sliding mat reduces friction and enables the caregiver to transfer users on practically any surface with a very simple hand grip and using a minimum of strength. Without unnecessary lifting, EasySlide provides a gentle and comfortable transfer for the user as well as for health care professionals.

After all these years, EasySlide is still the perfect and preferred assistive device for safe and easy transfers and for positioning in bed.



For supine transfers - EasySlide 01-1030



For positioning and supine transfers - EasySlide 01-1220

Easy transfers, both for the user and for the caregiver.

EasySlide (190 x 60) is primarily used for supine transfers, for example between beds and x-ray or operating tables, shower trolleys, stretchers and similar. But EasySlide is also a very effective aid for turning users in bed or on x-ray or operating tables.

Our own, unique sliding material, ULF (Ultra Low Friction), on the inner surface of the sliding mat reduces friction and enables the caregiver to transfer users on practically any surface with a very simple hand grip and using a minimum of strength. Without unnecessary lifting, EasySlide provides a gentle and comfortable transfer for the user as well as for health care professionals.

- Soft, padded outside provides for comfortable and gentle transfers.
- The ULF sliding material reduces the friction under the user.
- Protective and disposable covers are available as accessories.
- Removable protective cover containing carbon fiber reduces static electricity.

Being able to turn in bed improves the quality of life.

EasySlide (100 x 70) is mainly designed for users requiring only some care who need to turn and move in bed. It performs best on a rough surface. EasySlide reduces the friction on both shoulder and hip, which in turn reduces the risk of pressure sores. Thanks to these features, EasySlide is very gentle for people in pain but also for the carer, thus avoiding the risks associated with lifting. EasySlide promotes independence and improves the quality of life.

- Soft, padded outside provides for comfortable and gentle transfers.
- The ULF sliding material reduces the friction under the user.
- Protective and disposable covers are available as accessories.
- Removable protective cover containing carbon fiber reduces static electricity.





For positioning - EasySlide 01-1430



For positioning and sitting transfers - EasySlide 01-2020

Every pressure sore we can avoid makes life easier for bedridden people.

EasySlide (140 x 90) is designed for normal-weight users with limited to severely reduced abilities to participate in the turning process. These people are often permanently confined to the bed, and it is therefore important to prevent pressure sores. EasySlide ensures this by reducing the friction on the shoulders, hips and head during turning and by making it easy to turn the user as often as required. EasySlide is very gentle for people in pain. Furthermore, the load on the carer is reduced due to easier and less heavy transfers.

EasySlide (200 x 90) has extra padding in order to further reduce the risks of pressure sores. The padding distributes the pressure over the entire body and reduces pressure, friction and the risk of shearing on the especially vulnerable pressure points: the shoulders, hips, head, and feet. This version of EasySlide is mainly designed for users who are bedridden, with severely reduced functional abilities, often in intensive care units. By reducing the friction underneath the user, turning is performed easily and painlessly for both user and carer.

- The outside is made of microfiber, which are gentle against the body. This is especially important for bedridden users and for people in pain.
- The inside is made of our unique sliding material ULF (Ultra Low Friction), which reduces the friction under the user, thus very little force is required for the turning process.
- These are soft, comfortable and light materials which can be left permanently in the bed.
- Straps for fastening the sliding mat guarantee that it lies correctly in the bed.
- Good hygiene is a factor which also contributes to the prevention of pressure sores, and protective and disposable covers are therefore available as accessories.
- Removable protective cover containing carbon fiber reduces static electricity.
- Available in two lengths: 140 cm/56" and 200 cm/80".



For positioning - EasySlide 01-1470

A remarkably compact device that opens surprising opportunities.

EasySlide (50 x 60), (50 x 70) and (25 x 60) are excellent aids for turning in bed, for helping users higher up in bed, for helping them out of bed or for transferring them from the bed to the wheelchair. EasySlide is a small, practical aid that encourages activation. For the carer, EasySlide eliminates unnecessary lifting, which spares both back and shoulders. EasySlide is also an outstanding device for helping pregnant women with pelvic and back pain and people with reduced functional abilities to turn in bed independently.






- Soft, padded outside provides for comfortable and gentle transfers.
- The outer fabric is available in two types: polyester and microfiber. The polyester is less sensitive to dirt, while the microfiber is more comfortable when the user is lying on the product for long periods of time.
- The ULF sliding material reduces the friction under the user.
- Protective and disposable covers are available as accessories for EasySlide (50 x 60) and (50 x 70).
- Removable protective cover containing carbon fiber reduces static electricity.
- Available in three sizes.






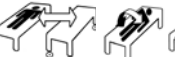



EasySlide helps users move higher up in bed.

QUICK REFERENCE

EasySlide is available in several different versions and sizes for different transfer situations and user needs. EasySlide is made of microfiber (polyester in 01-2010, 01- 2011, 01- 2015) with a padding of polyester and ULF (Ultra Low Friction), our own, unique sliding material. Most sizes can be fitted with a replaceable protective cover, which is made of polyester and can easily be cleaned. Disposable plastic covers provide for even better hygiene and less laundry. All in all, after all these years, EasySlide is still the perfect and preferred assistive device for safe and easy transfers and for positioning in bed.

	Art. No.	Product	Description
 <p>EasySlide 01- 1030</p>	01-1020	EasySlide	190 x 60 cm/76" x 24"
	01-1030	EasySlide (01-1020 + 01-1051)	190 x 60 cm/76 x 24", complete
	01-1051	Protective cover	For 01-1020, 011030
	01-1055	Disposable cover	For 01-1020, 01-1030, 2 x 50 pcs
	01-1151	Protective cover	For 01-1120*, 01-1130*
	01-1155	Disposable cover	For 01-01-1120*, 1130*, 2 x 90 pcs
		   	
			* 01-1120, 01-1130, EasySlide (100 x 60), discontinued

	Art. No.	Product	Description
 <p>EasySlide 01-1220</p>  <p>EasySlide 01-1430</p>  <p>EasySlide 01-1470</p>	01-1220	EasySlide	100 x 70 cm/40" x 28"
	01-1230	EasySlide (01-1220 + 01-1251)	100 x 70 cm/40" x 28", complete
	01-1420	EasySlide	140 x 90 cm/56" x 36", 2 fastening straps
	01-1430	EasySlide (01-1420 + 01- 1451)	140 x 90 cm/56" x 36", 2 fastening straps, complete
	01-1470	EasySlide	200 x 90 cm/80" x 36", 4 fastening straps
	01-1480	EasySlide (01-1470 + 01- 1481)	200 x 90 cm/80" x 36", 4 fastening straps, complete
	01-1251	Protective cover	For 01-1220, 01-1230
	01-1255	Disposable cover	For 01-1220, 01-1230, 2 x 90 pcs
	01-1451	Protective cover	For 01-1420, 01-1430
	01-1481	Protective cover	For 01-1470, 01-1480
01-1599	LockingStrap	Fastening straps, 1 pair, sparepart for 01-1420, 01-1430,01- 1470, 01- 1480	
		   	

	Art. No.	Product	Description
 <p>EasySlide 01--2015</p>  <p>EasySlide 01-2020</p>	01-2010	EasySlide	50 x 60 cm/19.7" x 23.6 ", polyester
	01-2020	EasySlide	50 x 60 cm/19.7" x 23.6 ", microfiber
	01-2030	EasySlide (01-2020 + 01- 2051)	50 x 60 cm/19.7" x 23.6 ", microfiber, complete
	01-2011	EasySlide	50 x 70 cm/19.7" x 27.6", polyester
	01-2021	EasySlide	50 x 70 cm/19.7" x 27.6", microfiber
	01-2015	EasySlide	25 x 60 cm/9.8" x 23.6 ", polyester
	01-2051	Protective cover	For 01-2010 01- 2020, 01-2030
	01-2052	Protective cover	For 01-2011, 01- 2021
	01-2055	Disposable cover	For 01- 2010, 01-2020, 01-2030, 100 pcs
			   

Accessories



01-1085 WallRack 75 x 35 x 17 cm/30" x 14" x 6.8"
 WallRack provides convenient storing of sliding mats and disposable covers.



P O Box 4791 Ferndown Dorset BH22 9WB
 Tel: 07857 603031 Fax: 01202 877240
 Email: info@amilakesouthern.co.uk

SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.

Transfer, between two locations. **Positioning**, within the same location.

Support, for mobility. **Lifting**, both manual and mechanical.
 mechainare.com