



REF
01-2090



MoveMaster is used for transfers between bed and wheelchair/portable toilet. MoveMaster is an open slide mat with three handles, one of which is longer than the other two. The different handles and the open design enable alternative working stances for the caregiver.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.




Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Functional inspection


Check the seams and strength of the MoveMaster by pulling on the handles. Test MoveMaster's friction by rolling it and noting how easily it glides. If possible, compare with a new MoveMaster.

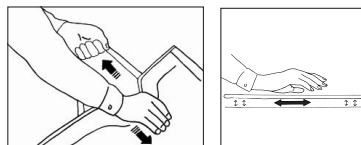

Always read the manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product. Do not leave the patient unattended during a transfer situation.

Using MoveMaster

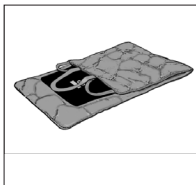

MoveMaster is a friction-reducing assistive device that can be used to advantage together with an EasyGlide to facilitate transfers in a seated position. MoveMaster is folded double and slides against itself to move the patient over the underlying surface.



Place MoveMaster under the patient using either one hand or with the help of an EasyGlide. EasyGlide can function as a bridge between the bed/wheelchair/shower chair. Lean the patient slightly in the opposite direction when positioning the MoveMaster. During transfer, the patient is leaned in the direction of transfer to reduce the contact surface under the 'sit bones' (the lowest bony prominences or ischial tuberosities), thereby sliding more easily.



To remove the MoveMaster, lean the patient in the opposite direction, hold the upper section of the MoveMaster, and then pull the lower section in the other direction.

Material:

Polyamide, polyether

Care of the product:

Read the product label.



Do not use rinsing agent. To prolong product life, avoid tumble-drying.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Product information

Art. no	Productname	Size	Description
01-2090	MoveMaster	47 x 120 cm /18 x 47"	3 handles

SystemRoMedic™

For 25 years SystemRoMedic have applied themselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, to help people, has always been, and will continue to be, the driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk