



AutoSlide is used to aid transfers into and out of car seats. It consists of two sections with surfaces that slide against each other. AutoSlide has a tension strap for fastening at the back. There are handles at the sides for caregivers.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



## Functional inspection

---

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.



## Always read the manual

---


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.




## Functional inspection

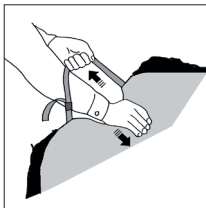

Test AutoSlide's friction by rolling it and noting how easily it glides. If possible, compare with a new AutoSlide.

Test clasp function and handles by applying tension to ensure that the product withstands heavy load.

## Placement of AutoSlide

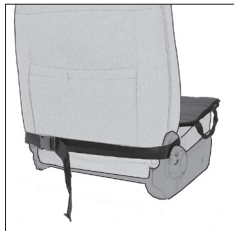

Place AutoSlide with the smaller section's rough surface facing the car seat cushion. Fasten the tension strap around the back support and tighten securely.

## Using AutoSlide



The patient can sit on the seat and then easily lift/turn the legs into the car while gliding all the way back onto the seat (1).

A caregiver can assist from the driver's side of the car by pulling on the handles on the sides of the AutoSlide (from the driver's side) (2).

NOTE! The patient should always wear a seatbelt.

## Material:

Polyester: Durable material with low friction.

## Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.



info@amilakesouthern.co.uk

# SystemRoMedic™

Easy-to-use assistive devices for

Easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

**Transfer:** products for moving patients between locations.

**Positioning:** products for repositioning in the same location.

**Support:** products that provide mobility support.

**Lifting:** products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, is to help people, has always been, and will continue to be, their driving force of innovation. They love easy transfers.



[info@amilakesouthern.co.uk](mailto:info@amilakesouthern.co.uk)