

AntiSlip provides friction under the patient's feet during different transfer situations, for example, when repositioning the patient higher up in bed or when raising from a seated to a standing position. AntiSlip is made of a material with closed cells, which makes it hygienic.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the manual


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.



Functional inspection

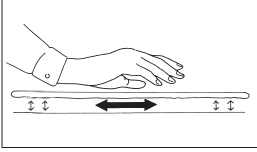


Test AntiSlip's anti-slip function against the intended underlying surface.

Placement of AntiSlip



AntiSlip is placed under the patient's feet or in the bed to provide more friction, thus enabling a better grip.

Using AntiSlip

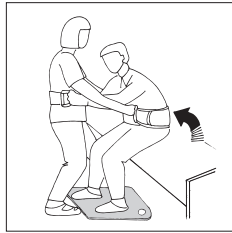

1. Provides friction under the feet in bed during pelvic lifting and for repositioning higher up in bed. Can be combined to advantage with a friction-reducing device under the shoulders.

2. AntiSlip can be used under the feet to provide a good grip during raising from a seated to a standing position or for transfers between different locations such as bed to wheelchair/chair. Use to advantage with EasyBelt/ FlexiBelt.

Care of the product:

Clean with dishwashing liquid or disinfectant. Wipe with a cloth.

Note: Do not subject the product to hard pressure or impact. This may leave indentations in the material.

Material:

Polyethylene



P O Box 4791 Ferndown Dorset BH22 9WB

Tel: 07857 603031 Fax: 01202 877240

Email: info@amilakesouthern.co.uk