


DisposableBelt

SystemRoMedic™

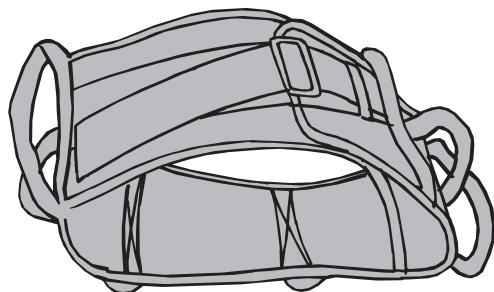

Manual - English



The DisposableBelt is intended for use as an accessory for extra support during various types of transfers and must not be used as an assistive device for lifting. The DisposableBelt is made of a very durable non-woven material. It must be discarded when soiled or when no longer required.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Do not launder. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

Placement of DisposableBelt



1. Apply the belt around the patient, below the waist.
2. Select the appropriate size loop. Fasten the belt and tighten it.

Removing DisposableBelt



Release the velcro and then remove the belt.

Using DisposableBelt



1




2

Caregivers can work in pairs or alone, depending on the patient's needs, since DisposableBelt have several vertical and horizontal handles. Cross-grips at the back provide support and enable a good working stance (1 and 2).



3

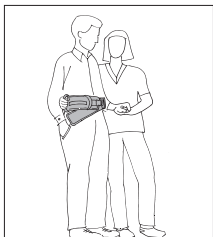

4

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the patient by the hand at the front (3).

DisposableBelt provide good support during transfers between wheelchair and bed (4).

Material:

Non-Woven, polypropylene

Care of the product:


Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.



Measurement:

	cm	Inch
Medium	84-94	33-37
Large	92-102	36.2-40.2

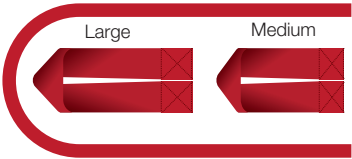

P O Box 4791 Ferndown Dorset BH22 9WB
Tel: 07857 603031 Fax: 01202 877240
Email: info@amilakesouthern.co.uk