



Max: 150kg/ 330lbs



HeelStrap is an accessory that can be used for heel support during transfers with the ReTurn7500/ReTurn7400. The length of HeelStrap is easily adjusted to fit the patient, and then secured with clasps. On the ReTurn7500/ReTurn7400 there are holes for fastening HeelStrap.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the product regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

Placement and use



Adjust the length of the strap to fit the patient. It should provide support behind the patient's heels. Place the heel strap behind the patient's heels and secure it on both sides using the clasps.

Care of the product:

Read the product label.



Do not use rinsing agent.
To prolong product life, avoid
tumble-drying.



P O Box 4791 Ferndown Dorset BH22 9WB
Tel: 07857 603031 Fax: 01202 877240
Email: info@amilakesouthern.co.uk