



The SlingBarWrap is an accessory that is placed around the slingbar. It protects the patient's head in the event of impact with the mid-section of the slingbar. The SlingBarWrap is suitable for patients who use slings that enable an upright seated posture, or for patients who have a tendency to tilt forward.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

Placement



Place the SlingBarWrap around the mid-section of the slingbar. The open section allows connection of the link from the lift arm.



Secure the SlingBarWrap by joining the Velcro strips.



Care of the product:

Read the product label.



Do not use rinsing agent.
To prolong product life, avoid
tumble-drying.

AMILAKE
SOUTHERN  LIMITED

P O Box 4791 Ferndown Dorset BH22 9WB

Tel: 07857 603031 Fax: 01202 877240 Email: info@amilakesouthern.co.uk