

ReTurnBelt

SystemRoMedic™



ReturnBelt	01-6031	XXS	01-6035	L
	01-6032	XS	01-6036	XL
	01-6033	S	01-6037	XXL
	01-6034	M	01-6038	XXXL



ReturnBelt Disposable	01-6233	S
	01-6234	M
	01-6235	L
	01-6236	XL



ReturnBelt Wipeable	01-6133	S
	01-6134	M
	01-6135	L
	01-6136	XL



SWL: 205 kg/450 lbs

ReTurnBelt is used for raising from a seated to a standing position, and for support in the standing position in combination with the use of ReTurn transfer platform. ReTurnBelt has a more rigid lower section for easy application and an elastic upper section for a better and more comfortable fit. ReTurnBelt has several gripping alternatives and a locking strap for extra support and safety in connection with the use of the ReTurn platform.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.



Always read the manual


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.




Functional testing

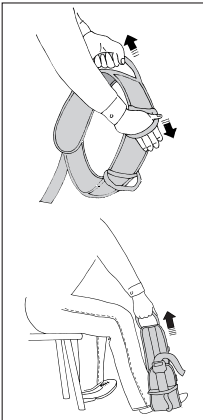

Pull hard on the handles in different directions.

Fasten the belt, place one foot in it, and then pull on one of the handles. Repeat the procedure with all handles.

Placement of ReTurnBelt

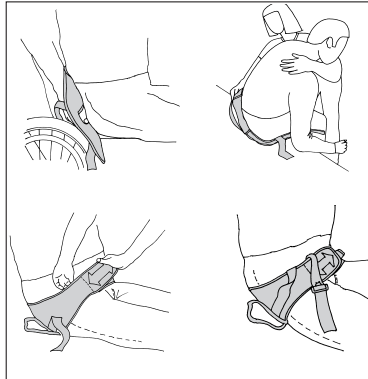

Fit the ReTurnBelt to the patient with the elastic section facing upward and the rigid, lower section downward, as shown in the illustration on the product label.

Lean the patient forward or to one side and guide the lower section of the belt down towards the 'sit bones' (lowest bony prominences or ischial tuberosities).

Secure the belt with the Velcro straps. The entire length of the front Velcro strap must be secured to the soft Velcro strap.


The locking strap can be temporarily folded under the edge of the belt, so that it is accessible when the patient has risen to a standing position. The locking strap is then looped around the ReTurn for added security while the patient is standing.

Using ReTurnBelt without ReTurn

The ReTurnBelt can also be used to advantage even without ReTurn, for example, for gait training. Make sure you use the right size and secure the both the Velcro straps and the clasp.

Using the ReTurnBelt

Two caregivers with ReTurnBelt

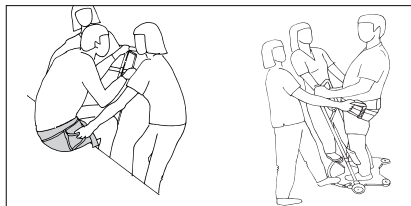


Brake the wheels on the ReTurn. The patient then leans forward and grasps the handles on the ReTurn. The caregivers then grasp the handles on the ReTurnBelt.

ReTurnBelt allows different gripping alternatives, and helps and encourages the patient to lean forward.

One caregiver helps the patient to lean forward and provides support during raising.

One caregiver with ReTurnBelt

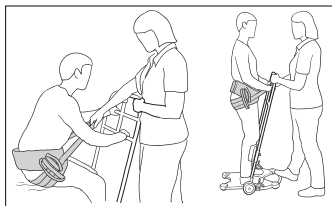


Fasten the clasp without tensioning the strap. A click is heard when the clasp is secured.

Brake the wheels on ReTurn.

The caregiver encourages the patient to lean forward while providing support during raising by grasping either a handle or the strap.

The patient leans forward and grasps the handles on ReTurn.

When it is necessary to attach ReTurnBelt to ReTurn7500

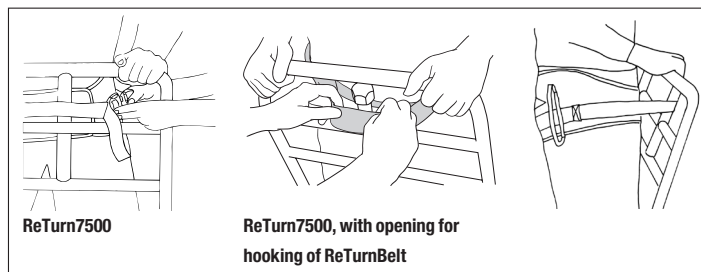

ReTurn7500:

Do not secure the clasp on the ReTurnBelt before rising. When the patient is in a standing position facing the ReTurn, the strap can be placed around one of the vertical handles. Secure the clasp on the ReTurnBelt. A click is heard when the clasp is secured.


ReTurn7500, with opening on the upper vertical handle:

Fasten the clasp on the ReTurnBelt without tensioning the strap. A click is heard when the clasp is secured.

When the patient is in a standing position facing the ReTurn, the strap is passed through the opening in the upper vertical handle and placed against the middle support bar.

Now, transfer the patient with ReTurn. Plan carefully to ensure that the transfer is as short as possible.

Sitting down

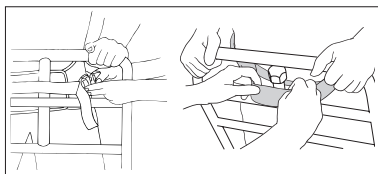

Brake the wheels on the ReTurn. Ensure that the patient is standing in a secure position facing the ReTurn, so that the locking strap is loosened. Undo the clasp and remove the strap from the ReTurn.

Fold it under the edge of the belt, so that it does not catch on anything.

Help the patient into a seated position with the help of the ReTurnBelt.

Material:

Polyamid, nylon

Disposable (non-woven): Polypropylene, hydrophobic

Wipeable: Nylon + TPU, possible to use disinfectant.

**Care of the product polamid/nylon:**

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

If disinfection is decided, 70% ethanol, 45% isopropanol or similar should be used.

Fasten the Velcro straps and clasp before laundering. Clean the Velcro straps to remove dust and dirt, so that they retain their adhesiveness.

Care of the product, non-woven:

Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

Care of the product, wipeable:

Read the product label.

Use ethanol (alcohol) to clean the cover. Concentration must not exceed one part ethanol to three parts water. Similar surface disinfectants may be used.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Product information

Art. no.	Model and size	Waist measurement, cm	Handles	Material, inside
01-6031	ReTurnBelt XXS	65-80	4	Polyamid/Nylon
01-6032	ReTurnBelt XS	80-90	4	Polyamid/Nylon
01-6033	ReTurnBelt S	90-105	4	Polyamid/Nylon
01-6034	ReTurnBelt M	105-125	4	Polyamid/Nylon
01-6035	ReTurnBelt L	125-145	4	Polyamid/Nylon
01-6036	ReTurnBelt XL	135-155	4	Polyamid/Nylon
01-6037	ReTurnBelt XXL	155-175	4	Polyamid/Nylon
01-6038	ReTurnBelt XXXL	175-195	4	Polyamid/Nylon
01-6233	ReTurnBelt S	90-105	2	Disposable
01-6234	ReTurnBelt M	105-125	2	Disposable
01-6235	ReTurnBelt L	125-145	2	Disposable
01-6236	ReTurnBelt XL	135-155	2	Disposable
01-6133	ReTurnBelt S	90-105	2	Wipeable
01-6134	ReTurnBelt M	105-125	2	Wipeable
01-6135	ReTurnBelt L	125-145	2	Wipeable
01-6136	ReTurnBelt XL	135-155	2	Wipeable

SystemRoMedic™

For 25 years SystemRoMedic have applied themselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, to help people, has always been, and will continue to be, the driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk