



REF

01-1510
01-1515
01-1520

SWL: 250-300 kg/550-660 lbs

TopSheet

SystemRoMedic™

TopSheet 1520, 1515 and 1510 are three versions of an incontinence sheet that supports the patient in a semi-recumbent position during transfers using a mobile or overhead lift. TopSheet consists of two sections that are held together by Velcro and can be easily separated. There are two suspension points, one on each side. TopSheet 1510 and 1515 feature an anti-friction fabric on the underside, which means they can be placed directly over the mattress cover or on a WendyLett without WendyLett2Way or 4Way. TopSheet 1520 is placed over TurnSheet 1590 or WendyLett4Way to minimize friction during turning.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.




Functional inspection

Visual inspection


Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Functional inspection

Test the function of clasps, loops and handles by applying tension to ensure that the product withstands heavy load.

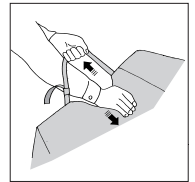


Note

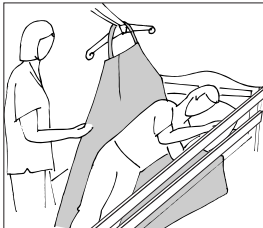


To ensure safety, a caregiver should stand on the side towards which the patient is being turned; alternatively, bed rails can be used. Never leave the patient unattended.

Lifting at a bias may entail risk for tipping. Ensure that the lift wheels can be moved laterally under the bed.

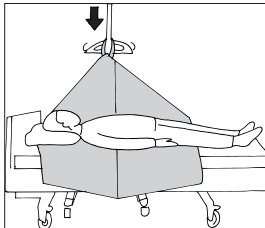

When a mobile lift is used, the slingbar should be placed directly over the suspension loop.

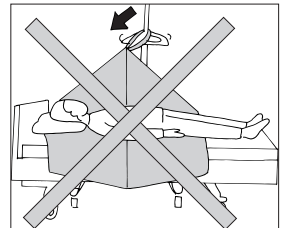

Lifting at a bias may entail risk for tipping. Ensure that the lift wheels can be moved laterally under the bed.


Placement




TopSheet 1520 is placed over TurnSheet 1580, 1590 or WendyLett4Way to minimize friction during turning. TopSheet 1510, which has a low-friction fabric on the underside, can be placed directly over the mattress cover or on WendyLett.

Read the manuals for other products used in combination with TopSheet.

Position the patient so that his/her hips are resting on the bottom section of the TopSheet. This makes it easier to change the sheet and enables easier access for cleaning and washing.

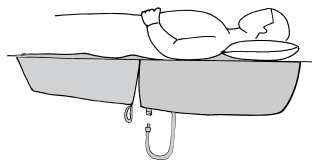

Turning the patient with TopSheet

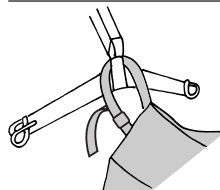


The TopSheet suspension loop is hooked above the slingbar on a mobile or overhead lift.

If bed rails are not used, a caregiver should stand beside the bed to ensure safety.


Start turn by raising the slingbar. The caregiver with the best view of the patient operates the lift.

Turn the patient to the desired position.

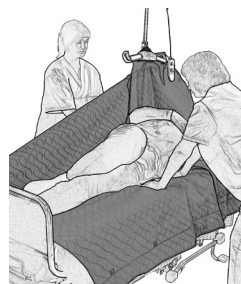

To ensure that the patient is repositioned in the centre of the bed, some manual adjustment towards the centre of the bed may be necessary during turning.

Turning and hygiene

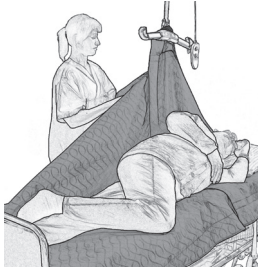

When the patient has been turned to the desired position, the Velcro strip can be loosened behind the patient.



Since the TopSheet now supports the patient's back, the caregiver is free to attend to the patient's hygiene.

Changing the bottom TopSheet section

Turn the patient on one side (see instructions for turning with TopSheet). Loosen the Velcro strip on the section on which the patient is lying and tuck it in as far as possible under the patient. Take a new bottom section, fasten it along the Velcro strip, and tuck it under the patient as far as possible.

Turn the patient back on to his/her back, move the lift to the other side, and turn the patient in the opposite direction. Remove the bottom section that is to be replaced. Pull the Velcro strip out and fasten the new bottom section to it.

Material:

Polyester, polyamid

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Size

Art no.	Productname	Size	Description
01-1510	TopSheet	200 x 160 cm/79 x 63 inch	SWL: 300 kg/660 lbs
01-1515	TopSheet	155 x 140 cm/61 x 55 inch	SWL: 250 kg/551 lbs
01-1520	TopSheet	140 x 140 cm/55 x 55 inch	SWL: 250 kg/551 lbs

Combination

Art no.	Combination
01-1510	Mattress cover 01-1593 or 01-1594 or WendyLett
01-1515	Mattress cover 01-1593 or 01-1594 or WendyLett
01-1520	TurnSheet and mattress cover 01-1590 or WendyLett + WendyLett4Way

Accessories

Art no.	
01-1509	140 x 160 (Lower part to 01-1510)
01-1519	80 x 140 (Lower part to 01-1520)
01-1514	85 x 140 (Lower part to 01-1515)

SystemRoMedic™

For 25 years SystemRoMedic have applied themselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, to help people, has always been, and will continue to be, the driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk