

TurnTableSoft

SystemRoMedic™



Manual - English




Ø 45 cm/18 inch
Ø 40 cm/16 inch.

REF

01-5086

01-5087

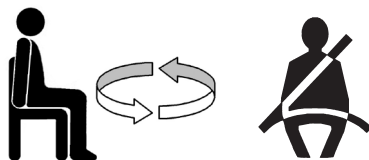

SWL: 150 kg/330 lbs



EN Always read the manual

Keep the manual where it is easily accessible for users of the product.
Do not leave the patient unattended during a transfer situation.

Manual no: 846 Ver. 11

AMILAKE
SOUTHERN LIMITED



TurnTableSoft

SystemRoMedic™

English

TurnTableSoft is used mainly for transfers to and from car seats, but also chairs and beds, as well as to facilitate rotating/turning. TurnTableSoft has two round discs with an anti-friction material inside that allows the discs to glide freely against each other. TurnTableSoft is available in two sizes.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.



Functional inspection

Visual inspection


Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

NOTE!

TurnTableSoft cannot be used on leather or rubberized upholstery.





Placement and use TurnTableSoft

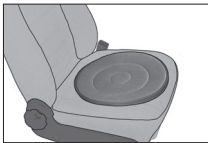


If used on a car seat, TurnTableSoft is placed on the middle of the seat.

When TurnTableSoft is used on a bed, it is placed near the edge of the bed at a level that allows the patient's head to rest on the pillow when he/she is lying down.

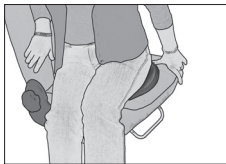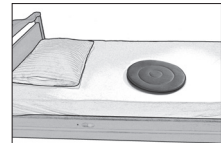

The patient sits at an angle on the car seat with legs outside of the car.

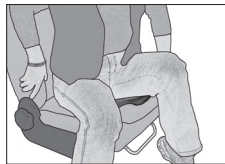

The patient then rotates easily while lifting his/her legs into the car.



The patient remains seated on the TurnTableSoft during travel. To get out of the car, the patient simply rotates to the side while lifting the legs out of the car. Always wear a seatbelt!

Material:

Polyamid, polyester

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Description

Art. No	Name	Description
01-5086	TurnTableSoft	Ø 45 cm/18 inch.
01-5087	TurnTableSoft	Ø 40 cm/16 inch.

SystemRoMedic™

For 25 years SystemRoMedic have applied themselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, to help people, has always been, and will continue to be, the driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk