

REF

01-6090



SWL: 150 kg/ 330 lbs

FlexiGrip is used as a helpful grip that facilitates more active raising of the patient from a recumbent to a seated position. FlexiGrip is fastened to the head of the bed and has several handles on both sides to enable the patient to 'climb' up to a seated position.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.




Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Functional inspection

Pull on the handles to ensure that they withstand a heavy load. If there are signs of wear, the product must be discarded.

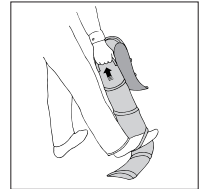

Always read the manual


Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.


Do not leave the patient unattended during a transfer situation.

Placement of FlexiGrip


Fasten FlexiGrip by drawing the straps through the loops. Check to ensure that the fixture points will withstand the pressure of the load transferred by the straps.

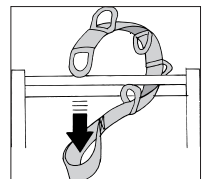

Using FlexiGrip

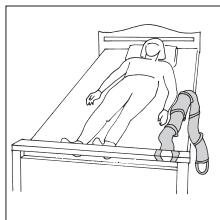


Leave FlexiGrip in the bed, so that it is easily accessible for the patient

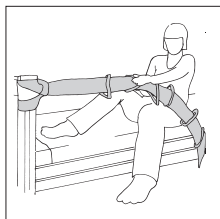

The patient can now 'climb' up to a sitting positioning with the help of FlexiGrip. The patient can also use FlexiGrip as a support for sitting on the edge of the bed.

To and from wheelchair with FlexiGrip

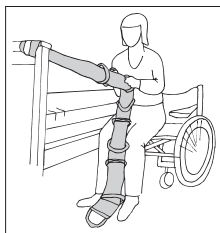

FlexiGrip can be used as support for transfers to and from a wheelchair. The patient can perform low-level repositioning by grasping the FlexiGrip, raising the buttocks and using the feet to manoeuvre into a seated position.

Material:

Polyester

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Product information

Art. no.	Product name	Size
01-6090	FlexiGrip	175 x 9 cm / 69 x 3,5" , 9 handles

Accessories

Art. no.	Product name
01-6091	Extension strap

SystemRoMedic™

For 25 years SystemRoMedic have applied themselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, to help people, has always been, and will continue to be, the driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk