


IncoSheet

SystemRoMedic



Manual - English

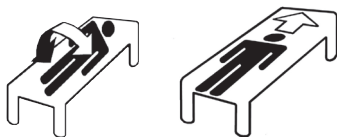

SWL: 300 kg/660 lbs



EN Always read the manual
Keep the manual where it is easily accessible for users of the product.

Product Ordering Information

Order ref	Product	Size	Description
01-1530	IncoSheet	75 x 90	
01-1531	IncoSheet	75 x 200	With fabric on both sides
01-1532	IncoSheet	75 x 150	
01-1533	IncoSheet	40 x 83	



info@amilakesouthern.co.uk

IncoSheet is a washable incontinence barrier that can be used in bed or on a wheelchair seat cushion. The material is highly absorbent and distributes fluid over a large area. The fabric on the top side of the product makes the surface feel dry and comfortable. The rough fabric on the underside provides friction to keep the sheet in place in the bed.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.






Functional inspection

Visual inspection


Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Placement

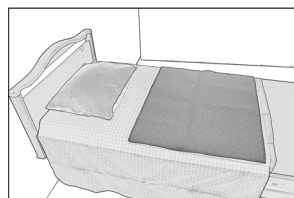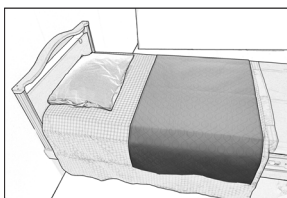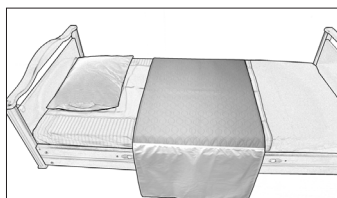


Place the IncoSheet with the quilted, lighter side facing upwards in the bed.

IncoSheet with the longer incontinence barrier is mainly intended for patients who need help with turning. This size model can be placed on top of the transfer assistive device, since it has an extra long incontinence barrier and continues to provide protection, even after turning.

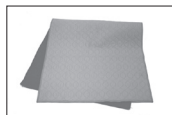

75x150 cm

The two shorter incontinence barriers are mainly intended for patients who are able to turn themselves, but they can also be used when the patient requires assistance to turn, in which case a transfer assistive device is placed on top. Since the transfer assistive device is placed on top of the IncoSheet, the IncoSheet remains in the same place under the patient during and after turning, thus continuing to provide incontinence protection.

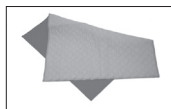


75x90 cm



75x200 cm

The small model, IncoSheet 40x83, can be placed in a wheelchair to protect the seat cushion.

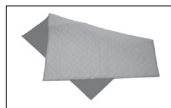

40x83 cm

Material:

Polyester, cotton

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Care of the product:

Read the product label.



IncoSheet can be laundered and dried at high temperatures.

Do not use laundry rinse or bleach (e.g., chlorine).

IncoSheet withstands more than 100 launderings.

SystemRoMedic™

Easy-to-use assistive devices for

Easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, is to help people, has always been, and will continue to be, their driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk