




Order Ref - 01-4005

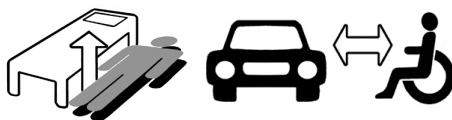

SWL: 120 kg/265 lbs

LiftSeat is a sling that is adapted for manual lifting. For the caregiver, it is easy to apply, and it gives the patient a feeling of security during manual lifting in a seated position. LiftSeat is soft and comfortable to sit on and it has a rough surface underneath, so that it can be left under the patient. LiftSeat is designed for use in most common situations where manual lifting in a seated position is required, such as transfers from wheelchair to aircraft seat, etc.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.


Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Functional testing

Pull on the handles to ensure that they withstand a heavy load. If there are signs of wear, the product must be discarded.

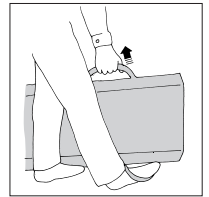

Always read the manual

Keep the manual where it is easily accessible for users of the product.

Do not leave the patient unattended during a transfer situation.

Placement of LiftSeat




LiftSeat is easy to place and remove. It can be pushed in under the patient from the side, or pulled down behind the back, and then forward under the patient's seat. Lean the patient to one side or forward.

Transfer or lifting



The handles provide a safe and comfortable grip, which makes it easier for caregivers to find a good working stance.

The lift can be performed either with one caregiver on each side, or with one in front and one behind.

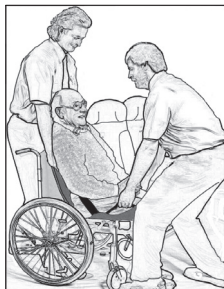

Lift with straight arms and a straight back. Bend your knees and lift with your legs!

Material:

Polyester

Care of the product:

Read the product label.



60-80° C
140-176° F

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

SystemRoMedic™

Easy-to-use assistive devices for

Easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Their mission, is to help people, has always been, and will continue to be, their driving force of innovation. They love easy transfers.



info@amilakesouthern.co.uk